Aesculap[®] Non-Stick Bipolar

Advanced Generator meets Advanced Instruments

Aesculap Neurosurgery

The Next Generation

Advanced Generator...

Bipolar Generator GN160

Combines ease of use and advanced technology for high efficiency from setup through the application.

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... meets Advanced Instruments

Rose Gold Bipolar Forceps

Rose Gold plated tips are characterized by a high conductivity and therefore offer very good non-stick behavior.

Disposable Rose Gold Bipolar Forceps

The disposable single use Rose Gold forceps are always new and ready to go and like the reusable Rose Gold forceps, have very good non-stick behavior.

BiProtect Bipolar Forceps

Ceramic insulated tips allow for particularly targeted and precise coagulation of the tissue.

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Aesculap® Bipolar Generator GN160

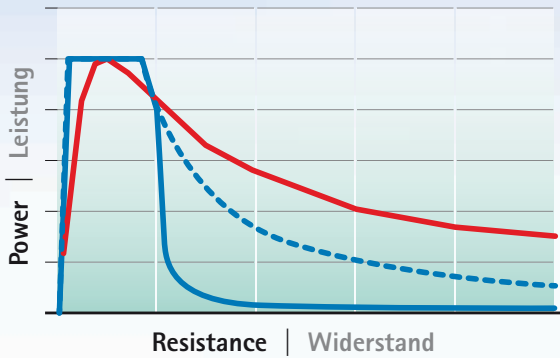
Technology that makes your life easier

Microprocessor-controlled electrosurgical unit suitable for applications in many disciplines.

- **High Tech for improved non-stick behavior**
 - Real-time monitoring and adaptation of coagulation parameters without adversely affecting the coagulation speed
 - Avoids sparking and therefore hot-spots
 - Also improves non-stick behavior for standard bipolar forceps
- **Ease of use**
 - Simple front panel
 - Simple knob to adjust power, easy switch to MALIS mode
 - Single-pedal footswitch to activate the unit (GN161 or GK226)
- **Compatibility and flexibility**
 - Compatible with Rose Gold and all other bipolar forceps
 - Compatible with Jet Irrigation Unit GN090

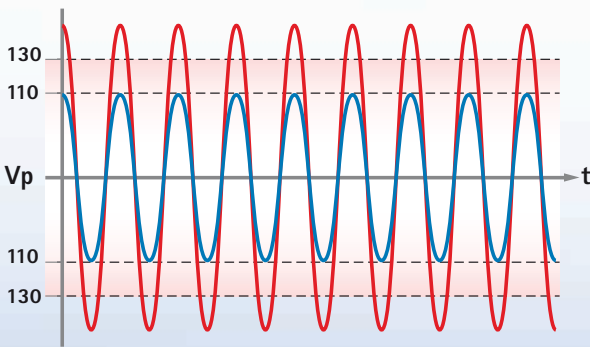
- GN160 Standard dose / Standard Dosis
- Active power reduction / Leistung wird aktiv reduziert
- Conventional generator eg. GN060 Standard dose / Herkömmlicher Generator z. B. GN060 Standard Dosis

The Aesculap Non-Stick Generator actively reduces the power when necessary to prevent carbonization and sticking of the tissue.

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Micro sparks are one cause of sticking. They can occur at a voltage of 130 Vp and higher. The new bipolar generator works at a max. voltage of 110 Vp (in standard mode) and therefore prevents the occurrence of micro sparks.

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Aesculap® Rose Gold Bipolar Forceps

The new Aesculap "Gold Standard" for non-stick forceps

These forceps stand out for their reduced tissue adhesion. The Rose Gold plated tips allow for a high heat conductivity and a high electrical conductivity. This combination enables quick heat dispersion over and away from the tips which makes a very good non-stick behavior possible.

- Less tip cleaning
- Less procedural interruptions

In addition the slender design of these forceps provides a clear view of the surgical operative site.

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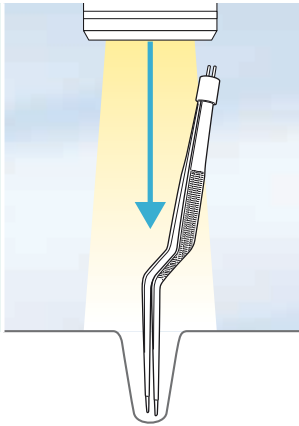
Angled bayonet shape

For enhanced sight and easier handling. It moves the surgeons hand out of the sight while working under the microscope.

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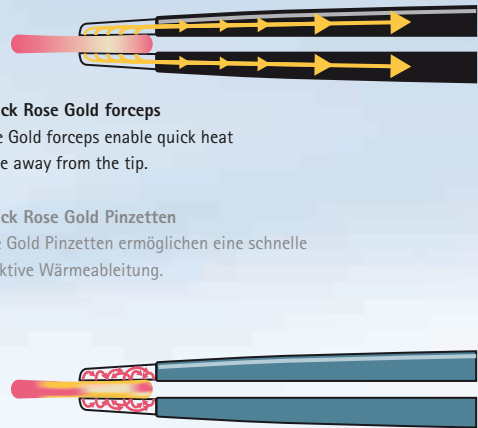
Non-Stick Rose Gold forceps
The Rose Gold forceps enable quick heat discharge away from the tip.

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Standard forceps
The heat can not be discharged fast enough. This leads to overheating and sticking of tissue.

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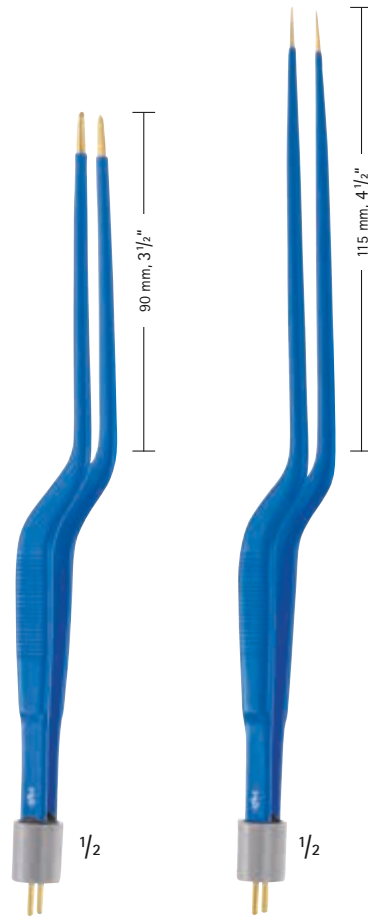
Aesculap® Rose Gold Disposable Bipolar Forceps

The new Aesculap "Gold Standard" for disposable non-stick forceps

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| | | |
|--|----------------|----------------|
| 0.5 mm  1/1 | GK501SU | GK504SU |
| 1.0 mm  1/1 | GK502SU | GK505SU |
| 1.5 mm  1/1 | GK503SU | GK506SU |
| Total Length / Gesamtlänge | 190 mm, 7 1/2" | 215 mm, 8 1/2" |

Using a brand new disposable forceps for every case means consistent coagulation quality

- Sterile
- Always ready to use
- No reprocessing

Rose Gold plated tips reduce tissue adhesion

- Less tip cleaning
- Less OR interruptions

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Aesculap Neurosurgery

Stainless steel bipolar forceps

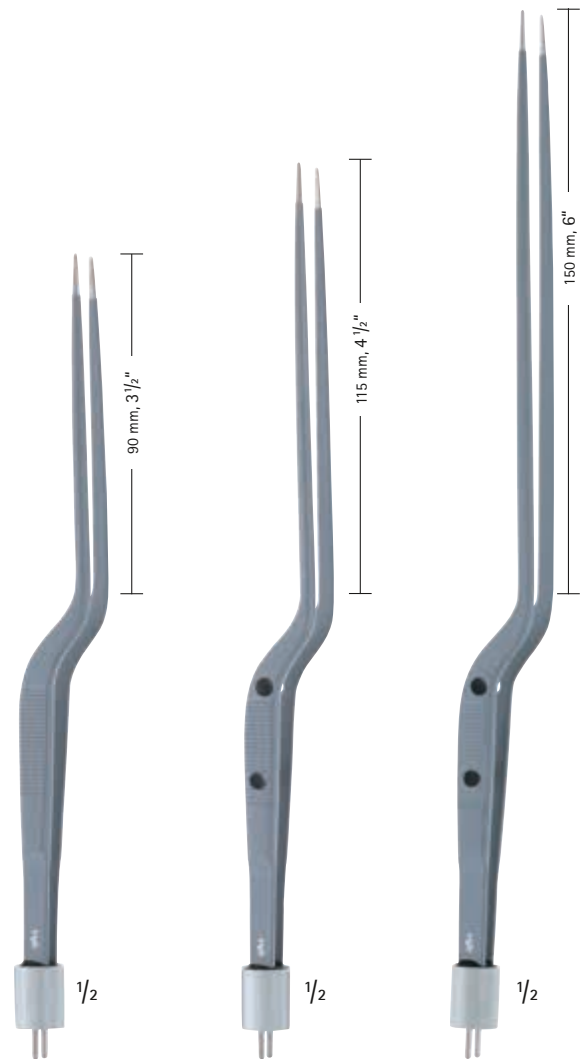
Clear view on the surgical operative site

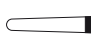
Our new stainless steel bipolar forceps with the same slender design as our new Rose Gold line is the basic version and is an alternative at a favorable price.

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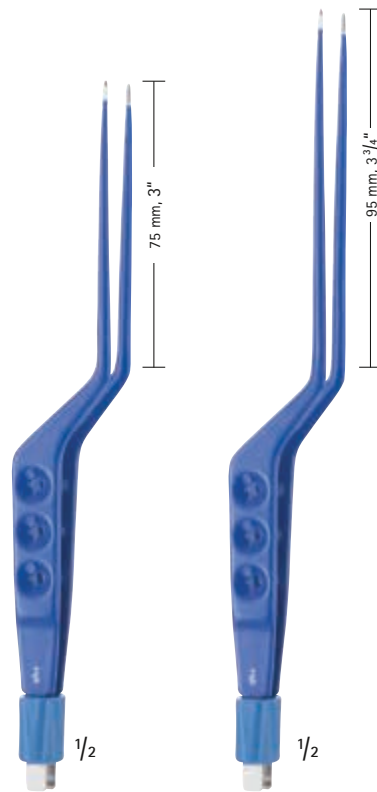
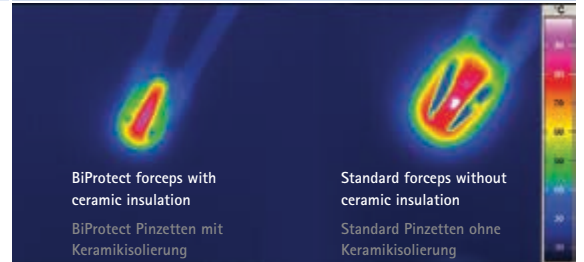
Aesculap® BiProtect Bipolar Forceps

Anti Collateral Damage - Advanced Precision

 Aesculap tab /
Aesculap Flachstecker

| | | |
|--|----------------|----------------|
| 0.4 mm  1/1 | GL470R | GL480R |
| 0.8 mm  1/1 | GL471R | GL481R |
| Total Length / Gesamtlänge | 195 mm, 7 3/4" | 215 mm, 8 1/2" |

BiProtect

Ceramic is an excellent insulator. The ceramic insulation surrounding the outer instrument tip effectuates a heat concentration on the inner and distal part of the forceps.

This is especially helpful when working in narrow spaces surrounded by sensitive structures which need to be preserved.

Together with the fine instrument tips BiProtect is the choice for particularly targeted and precise coagulation.

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New Bipolar Cables

Suitable reusable and disposable cables

| Reusable cables Wiederverwendbare Kabel | | Equipment side Geräteseitig | Length / Länge: 4 m | Instrument side - universal connector Instrumentenseitig - Universal Stecker |
|--|--|--------------------------------|---------------------------|---|
| GN132 | ERBE int. | 22 mm | | Aesculap round pin Aesculap tab US round pin |
| GN133 | Aesculap GN160 BOWA Lamidey Söring Valleylab | 28.6 mm | | Aesculap round pin Aesculap tab US round pin |
| GN134 | Codman Synergy | 38 mm | | Aesculap round pin Aesculap tab US round pin |
| GN131 | Erbe Storz | 8/4 mm | | Aesculap round pin Aesculap tab US round pin |
| GN130 | Aesculap GN160 ACMI Berchtold Martin Wolf | 5/2 mm | | Aesculap round pin Aesculap tab US round pin |
| Disposable cables Einwegkabel | | Equipment side Geräteseitig | Length / Länge: 3,70 m | Instrument side - universal connector Instrumentenseitig - Universal Stecker |
| GK290SU | ERBE int. | 22 mm | | Aesculap tab US round pin |
| GK291SU | Aesculap GN160 BOWA Lamidey Söring Valleylab | 28.6 mm | | Aesculap tab US round pin |
| GK292SU | Codman CMC I CMC II* | 38 mm | | Aesculap tab US round pin |

For more information see Brochure C30481.

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